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INTENDED USE (A-110, A-120)

To occlude common medical grade tubing with an outside diameter of not less than 1mm or more than 8mm.

DIRECTIONS FOR USE (A-110, A-120)

(1) Make sure jaws grip tubing completely. (2) Press clamp together until tubing is fully occluded and the locking ratchet is engaged. (3) Observe clamp and tubing while in use to determine clamp stability and proper occlusion.

CLEANING INSTRUCTIONS (A-110, A-120)

(1) Use a warm, soapy water to clean after use, rinse and dry. (2) As an alternative, use your facility standard cleaning procedure.

INTENDED USE (T-200)

To secure towels, drapes, and sponges during non-invasive procedures.

DIRECTIONS FOR USE (T-200)

(1) Make sure jaws grip drapes, towels and/or sponges completely; and (2) Clamp together until the item is fully secured and the locking ratchet is engaged.

WARNINGS AND CAUTIONS

(1) Do not freeze or subject clamp to temperatures exceeding 132°C. (2) When cleaning either clamp follow your own internal procedures in order to prevent over exposure to chemicals during the cleaning process. In some cases, the manufacturers of cold sterilants set exposure limits that may be included in your internal procedures. (3) When cleaning or sterilizing, it is suggested that the clamp be in the open position. (4) The clamp will suffer fatigue and discoloration from repeated use, cleaning and/or sterilization.

LIMITED WARRANTY (A-110, A-120 or T-200)

Dravon warrants that each A-110, A-120 or T-200 clamp is free from defects in materials and workmanship. Repeated reuse, which requires cleaning and/or sterilization is the responsibility of the user.

DRAVON MEDICAL, INC.
11465 S.E. HIGHWAY 212
CLACKAMAS, OREGON (USA) 97015-9079
Telephone: (503) 656-6600 FAX: (503) 655-5229
USA/Canada Only 1-800-654-1976/E-Mail:sales@dravon.com

EC REP Advena Ltd. Tower Business Centre, 2nd Floor
Tower Street, Swatar, BKR 4013 Malta **CE**

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