

Directions For Use
Dravon T-Clamp Catalog Number T-200
Non-Invasive, Sterile or Non-Sterile Clamp

INTENDED USE OF T-200, T-200-2 and T-200-5

To secure drapes, towels, and/or sponges.

DIRECTIONS FOR USE

- (1) Make sure jaws grip drapes, towels and/or sponge completely.
- (2) Press clamps together until the item is fully secured and the locking ratchet is engaged.
- (3) Do not attempt to force clamp beyond the last lock notch on the clamp.
- (4) Observe the clamp and the item being secured to determine grip stability.

STERILIZATION PROCEDURE

This product is supplied non-sterile (T-200) or sterile (T-200-2 or T-200-5). If non-sterile, the T-Clamp may be sterilized. Dravon maintains test data related to the autoclave and EtO method of sterilization on file.

WARNINGS AND CAUTIONS

- (1) Do not freeze or subject clamp to temperatures exceeding 132°C.
- (2) When cleaning or disinfecting this clamp, follow your own internal procedures in order to prevent over exposure to chemicals. In some cases, the manufacturers of cold sterilants set exposure limits that may be included in your internal procedures.
- (3) When cleaning/disinfecting or sterilizing, it is suggested that the clamp be in the open position.
- (4) With any plastic device, repeated exposure to heat and chemicals will cause a loss of clamp strength. You must remain alert for any weakness that may occur.

LIMITED WARRANTY

Dravon warrants that each T-200, T-200-2 or T-200-5 Clamp is free from defects in materials and workmanship. This clamp is intended for use on drapes, towels, and sponges at operating temperatures above freezing. Reuse, which requires cleaning/disinfecting and/or sterilization is the responsibility of the user.

DRAVON MEDICAL, INC.

11465 S.E. HWY 212

CLACKAMAS, OREGON (USA) 97015

TELEPHONE: (503) 656-6600

FAX: (503) 655-5229

USA/Canada Only: (800) 654-1976

E-Mail: sales@dravon.com

7.1.DM.0200.G 04/12